

Multi Drugs Test

• Biology

The multi drugs test can detect 4 groups of drugs: cannabis, amphetamines, cocaine and opiates.

Each number on the screening device is linked to a group of drugs.

C - Control

1 - Cannabis: 50ng/mL ($\Delta 9$ -Tetrahydrocannabinol)


2 - Opiates: 40 ng/mL (morphine)

3 - Amphetamines: 50 ng/mL (D-amphetamine)

4 - Cocaine: 20 ng/mL (cocaine)

• Content

- 1 pouch containing 1 screening device and its cap
- 1 instruction
- 1 support booklet

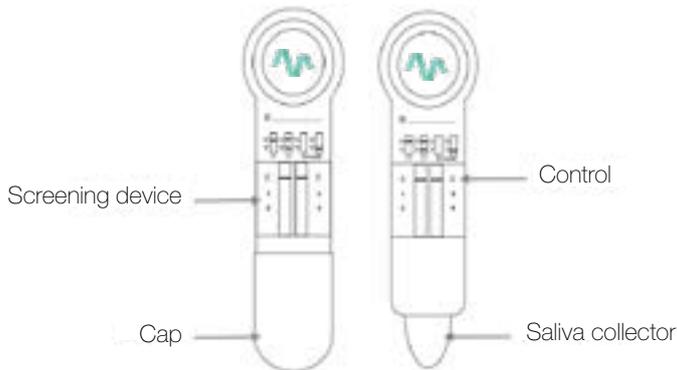


• Precautions

1. External use only.
2. Read carefully the instructions before having the test.
3. Keep between 2°C and 30°C. Do not deep-freeze.
4. Do not use after the expiry date or if the pouch is damaged.
5. Do not use the test again.
6. Keep away from children.
7. After use, throw all components away.

• Use


1. Do not eat, drink or smoke 10 minutes before the test.
2. Once the pouch is open you should do the test.
3. Place the saliva collector under your tongue. Hold it slightly tilted up (around 15°) and press constantly your tongue against the collector. Do not bite or chew the device.

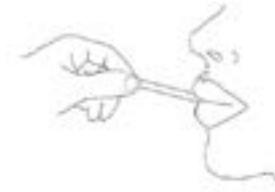

4. Keep it in your mouth until the pink migration appears (about 1 min).

Caution! do not remove before the pink migration appears or you might have an invalid test.

The pink migration appearance may vary according to people, it can last up to 10 min.

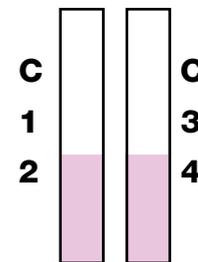

5. Close with the cap and leave it flat for 6-7 minutes. Then read results. Do not exceed 15 minutes.


• Results

POSITIVE

STRIP 1, 2, 3 or 4 NON VISIBLE

The test is positive when a strip appears next to C (Control) and not next to the numbers.

You should not do any action needing full attention or focus, such as driving.

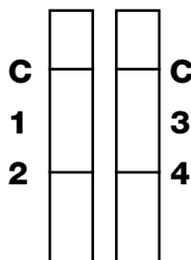

Positive to substances 1 and 3 &
negativ to substances 2 and 4

• Test limits

IMPORTANT

This screening test should be done at least 12 hours after using a psychoactive substance.

A negative result does not necessarily imply there is no psychoactive substance.

If their concentration is below the test threshold, it can appear in the saliva sample.

IF YOU HAVE ANY DOUBT, DO NOT DRIVE!


NEGATIVE

STRIP 1, 2, 3 or 4 VISIBLE

The test is negative when a strip appears next to C (Control) and also next to the numbers.

Intensity of strips may be lighter than C.

A strip next to 1, 2, 3 or 4, even very light is a negative result.

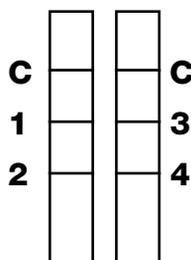

Negative to substances 1, 2, 3 and 4


INVALID

C STRIP NON VISIBLE

The test is invalid when there is no strip next to C (Control).

It often happens when the instructions are not completely respected.

Read the instructions and do another test.

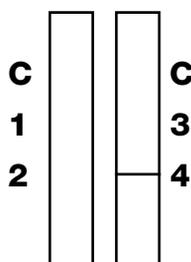

Invalid



contact@medisur.fr
+33 4 42 96 09 68
www.medisur.fr



MEDISUR
100 Impasse des Houillères
13590 Meyreuil, France

ref. MULTL_01
rev. 01/2020

**JEUNE ENTREPRISE
FRANÇAISE**